

# H6000 Handpiece cleaning and lubrication system User Manual



[www.tpcdental.com](http://www.tpcdental.com)

(800) 560-8222

[service@tpcdental.com](mailto:service@tpcdental.com)

# Installation instruction for the H6000 Handpiece cleaning and lubrication system.

1. Place the H6000 in a location where you have an air line. Generally this location will be located near your sterilization area. You will also need an electrical outlet next, Connect your air line and electrical.



2. Fill lubrication reservoir with the supplied lubrication fluid. Make sure not to over fill the reservoir or spillage will occur.



3. Place absorbent pads into place. These pads will absorb any excess lubrication that is released in the chamber. The pads need to be replaced every tank cycle.



# Operation Instructions for H6000 HandPiece lubrication System

1. Open the door and attach up to 3 handpieces to the H6000 ports.



2. After attaching handpieces close the door. The H6000 will not operate if the door is not properly closed.



3. Select the S (Short-35 second cycle), L (Long-45 second cycle) or S, L, E (Extra long-50 second cycle) for each individual handpiece.


- SHORT CYCLE  
For high speed handpieces
- LONG CYCLE  
For low speed handpieces & air motors
- EXTRA LONG CYCLE  
For speed reduction handpieces



4. Press the start button. In less than 3 minutes all handpieces are correctly cleaned and lubricated. (3 minutes for 3 handpieces)



5. Press the air button if you wish to run a cycle of only air through your handpieces. This process removes any excess lubrication fluid from the turbine.

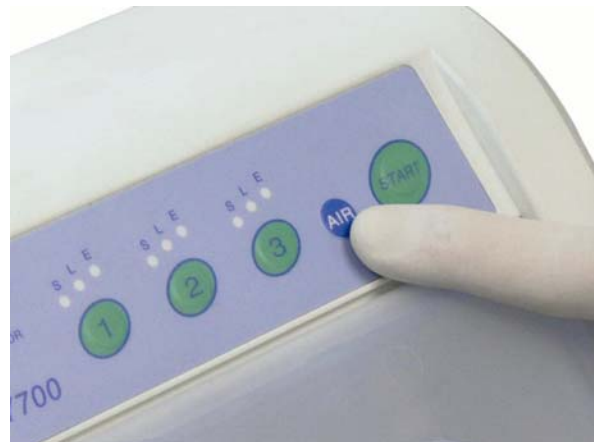

6. Remove your handpieces from the H6000. lubrication and cleaning is now complete.
7. You may proceed to sterilization (if necessary).



# Overview of H6000 Controls

